


SPLIT AUTOMIX CARTRIDGE GUN

Note: This cartridge gun is designed for split cartridges. The cartridges feature a gap between the two barrels.

1. Push up the release lever at the rear of the gun and pull the plunger as far back as it will go.



2. Flip up the cartridge lock and seat the cartridge in the gun. (The notch in the base of the cartridge should face downward as in the photo.)

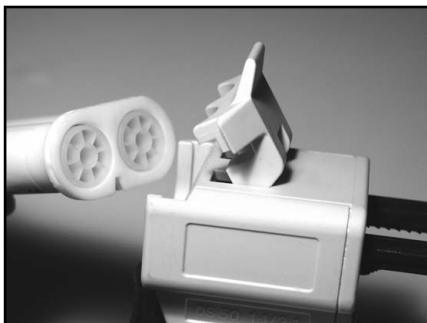

3. Close the cartridge lock.



4. Before affixing the mixing tip: Remove the cap that seals the cartridge by twisting the cap counterclockwise and pulling it from the cartridge. Discard the cap.

Express approximately 1/4" of material directly from the cartridge onto a pad to assure that both components are flowing properly.

5. To affix the mixing tip. Align the small triangle on the colored base of the tip with the corresponding notch in the cartridge. Seat the tip fully, and lock it in place by rotating the tip's colored base clockwise.

IMPORTANT: Never use excessive force when expressing material. Doing so may damage the gun, rupture the cartridge or cause the rear of the cartridge to leak. If the material does not flow easily, remove the tip and use an explorer to remove any set material clogging the cartridge orifices.

6. To remove the cartridge, push up the release lever and pull the black plunger as far back as it will go. Open the cartridge lock to release the cartridge.



ORDER TOLL-FREE **1.800.243.7446** USA & CANADA
631.249.1134 • Fax: 631.249.1242 • info@parkell.com
www.parkell.com